


Audiometry: Adding non-Otosuite Speech Material for Testing–Windows 7/10


This document describes how to “rip” a CD and save the speech files to the computer’s hard drive for use in Otosuite.

1. Ripping the CD


- Insert the CD into the CD drive of the computer. If the Auto Run window appears, click “Cancel”
- Open Windows Media Player software.
- Click on “Organize” and select “Options...”

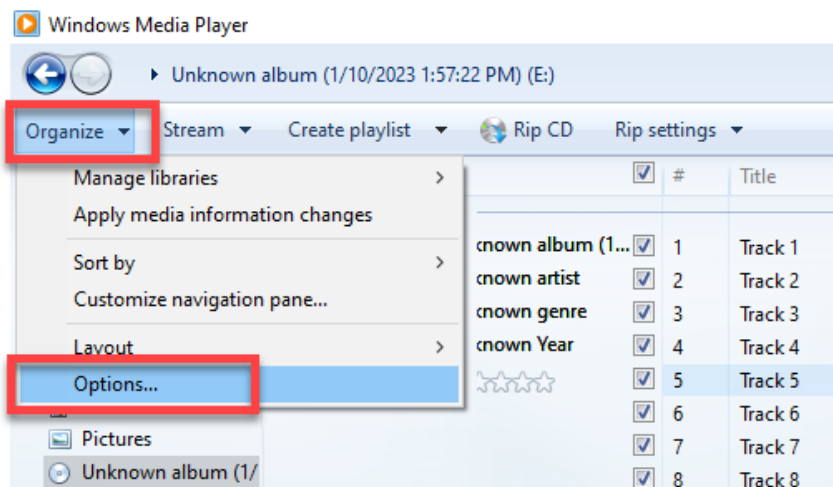


- Click on the “Rip Music” tab and “Change...” to specify where to save the speech files.

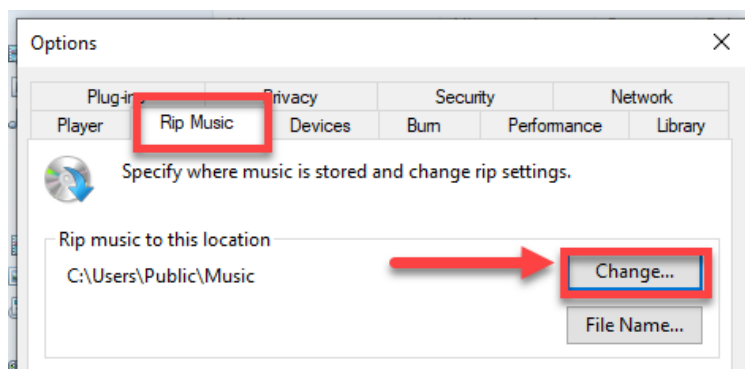


- e. In the Browse For Folder window, navigate to C:\Program Files (x86)\GN Otometrics\OTOSuite\WordLists\OTOSuite WordLists – US and click OK. (Note: If you do not have permissions to save to this location, save to another location such as a folder on the Desktop)

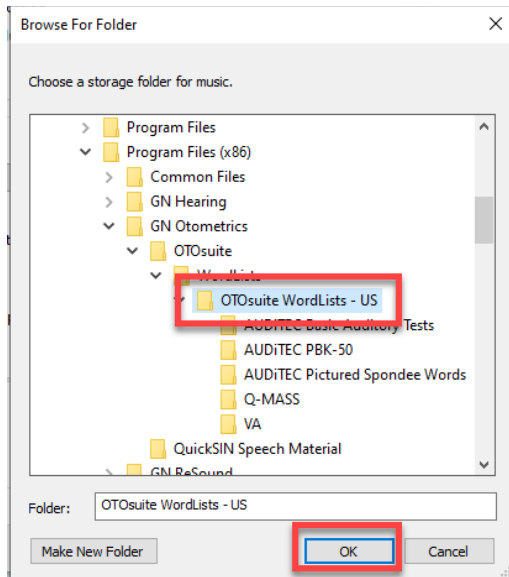


- f. On the “Rip Music” tab, select the Format: MP3 or WAV (larger file size), uncheck “Rip CD Automatically” and slide Audio quality to the right to “Best Quality”. Click Apply, then OK.

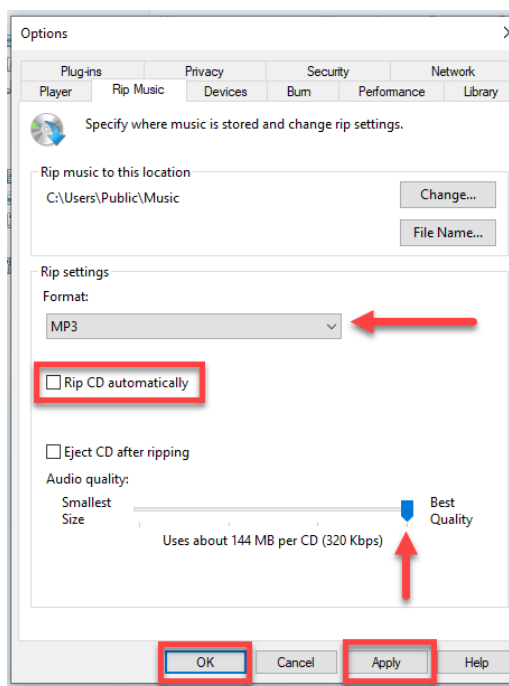


- g. Select “Unknown Album” by clicking on the arrow to the left of Library (if not already displayed)

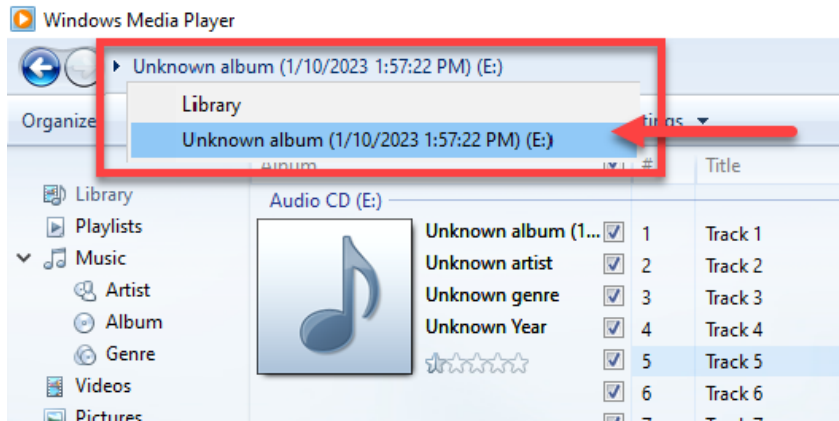

- h. Edit the Album, Artist and Track names as needed by right clicking on each of the items and selecting “Edit.” Edit the names as listed on the CD cover or insert. (Note: These must be re-named prior to ripping the CD.)

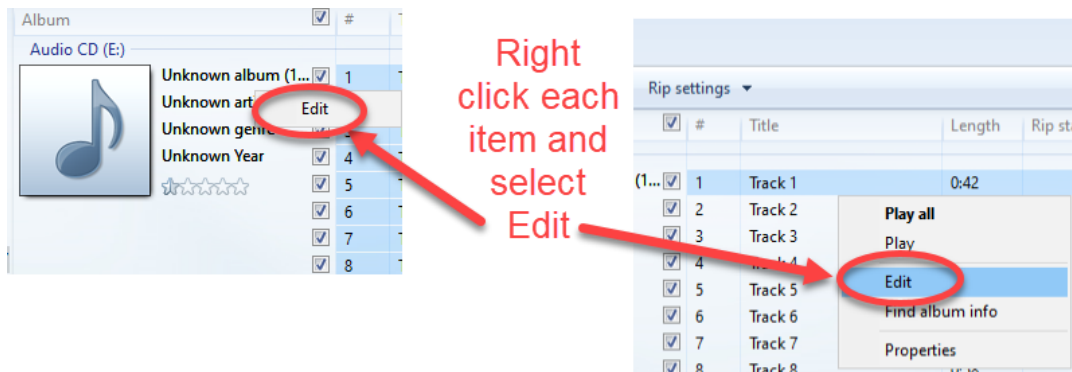


- i. Select “Rip CD” in the upper toolbar

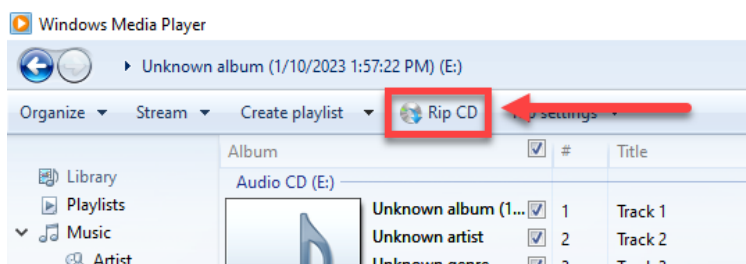

2. Playing the speech files in Otosuite

To play your new word list, open Otosuite and go to the Speech tab (example is showing the *Astera Speech* tab). Make sure Int. File is showing in the dropdown in the Channel Settings options. Click on the upper dropdown menu to select the speech file folder and select <Browse...> at the bottom of the list. When the Browse For Folder window opens, navigate to where you saved the speech files, select the folder and click OK. The lower dropdown should now display the tracks contained within that folder.

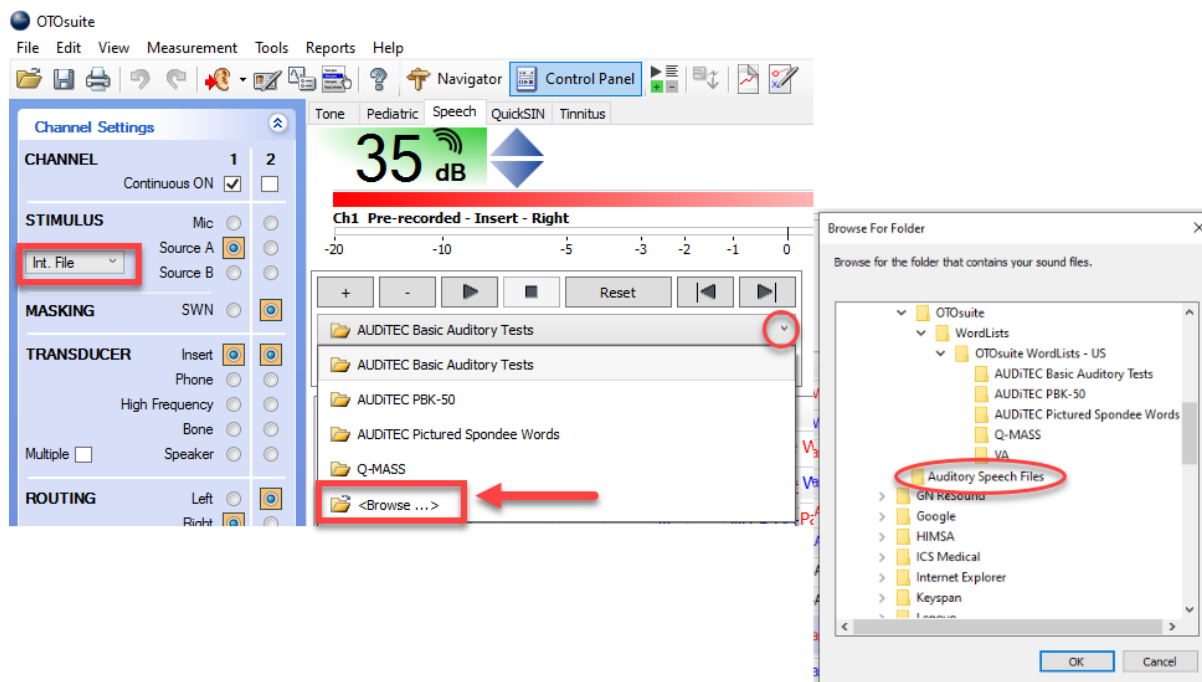

3. Save to a User Test for future use

You may want to save a new User Test to quickly access these speech files during future testing. Please see the Otosuite Speech Tab Setup and Otosuite User Test and Start-up Education Resources for instructions on how to save these changes.