


OAE and audiometry screening

▶ Turning On/Off, and charging the device

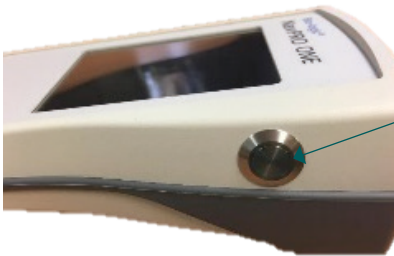


On/off button on right side



Power supply connector

▶ Start menu

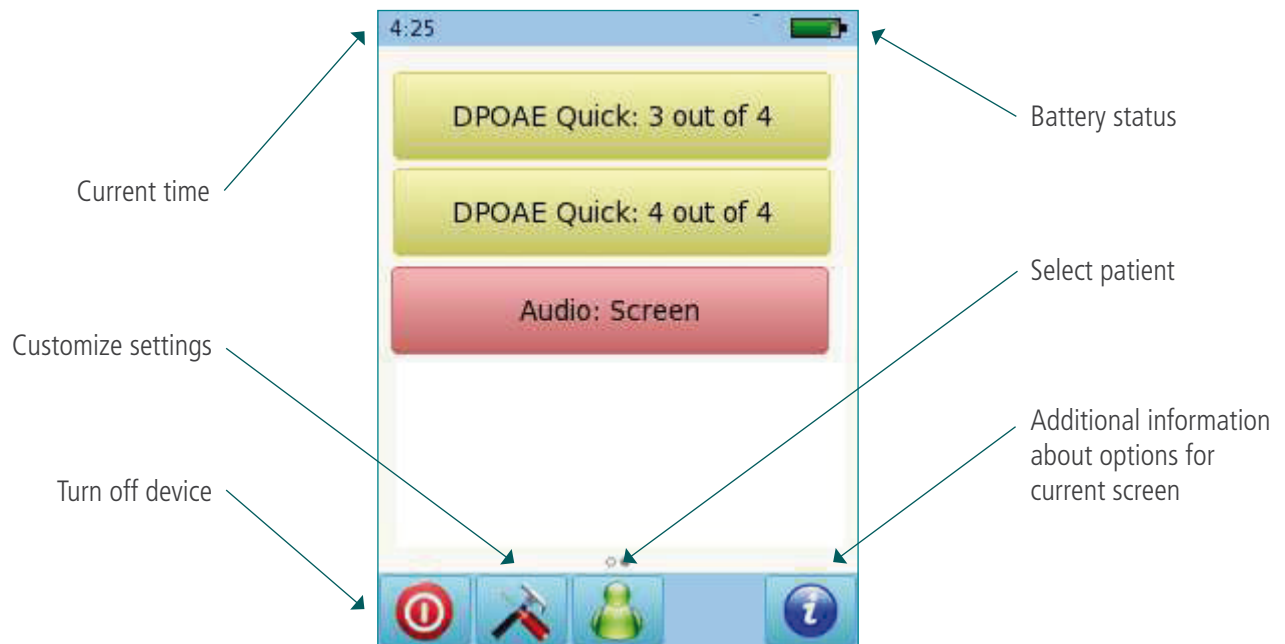


▶ Testing: Audiometry

Connect the headphones to the blue connector.
Select the Audiometry: Screening button from the start menu.
Select the ear on which you want to start the test.



Audiometry test screen

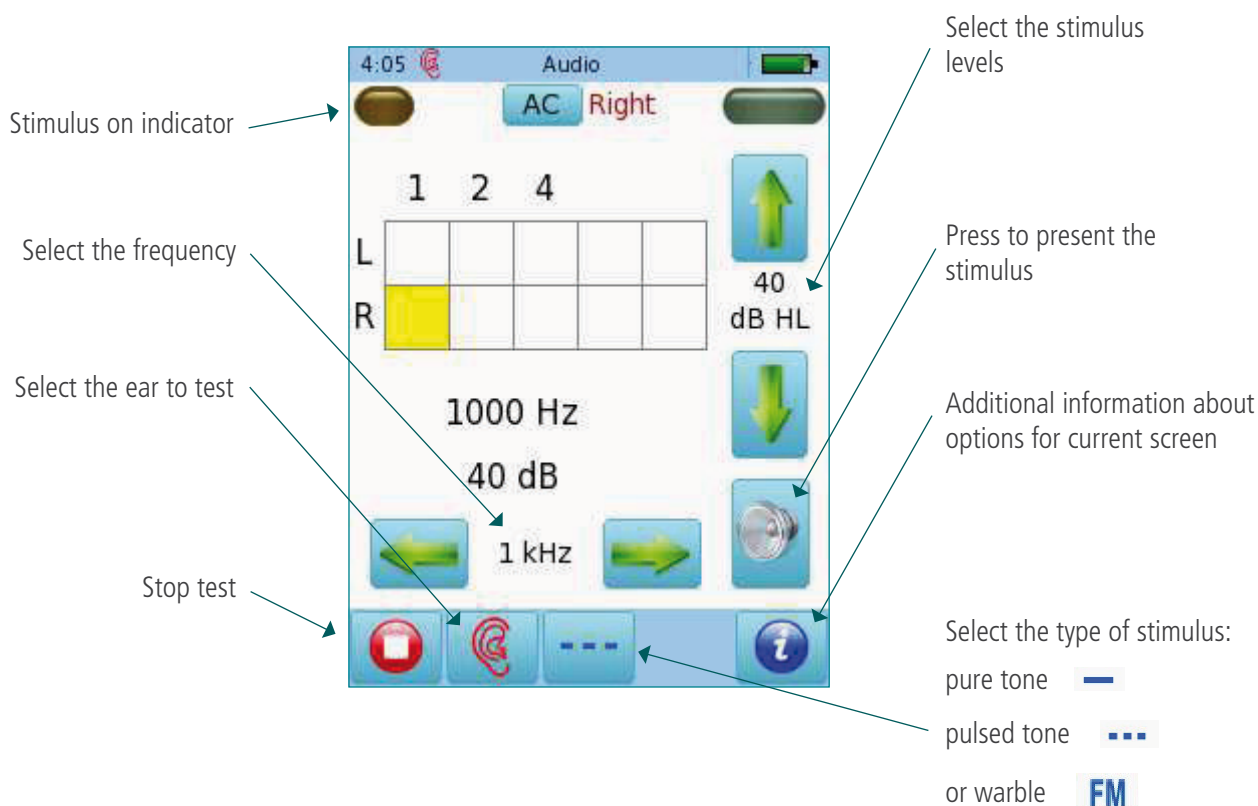


As you present the stimulus, the current level will be displayed on the corresponding (yellow) square. Once the hearing status for a frequency has been identified, move on to the next frequency.


Tapping twice on a square will display NR (no response).

Tap three times to get back to a blank square.

Once all frequencies have been tested for one ear, you can test the other ear.

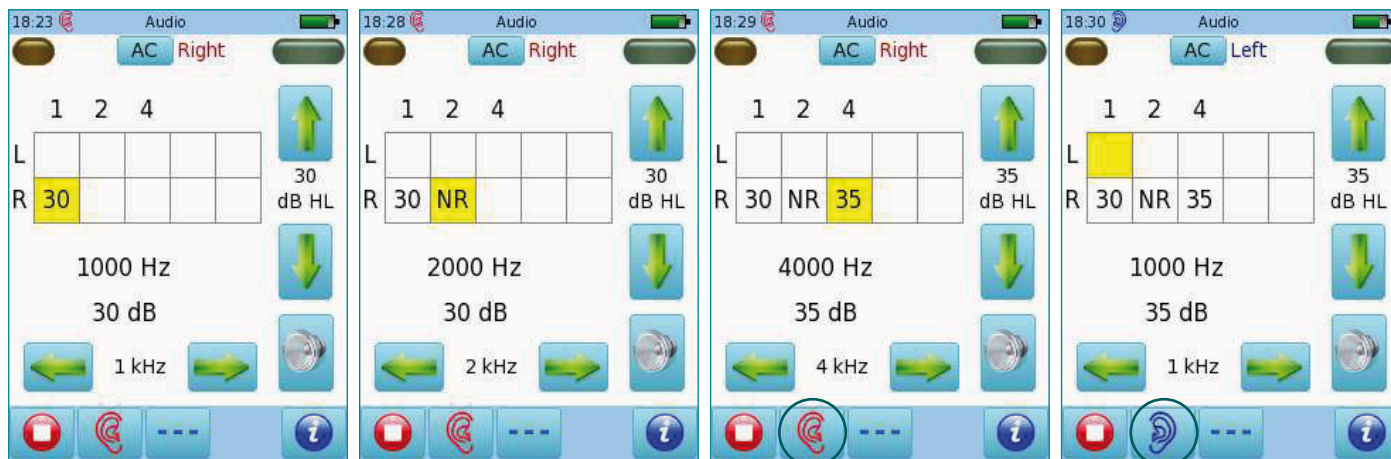


When the test is finished, press the Stop button. You can add the patient's personal data, and save on the device, print to a label (a label printer is required) or print to a pdf file or full page report on PC.

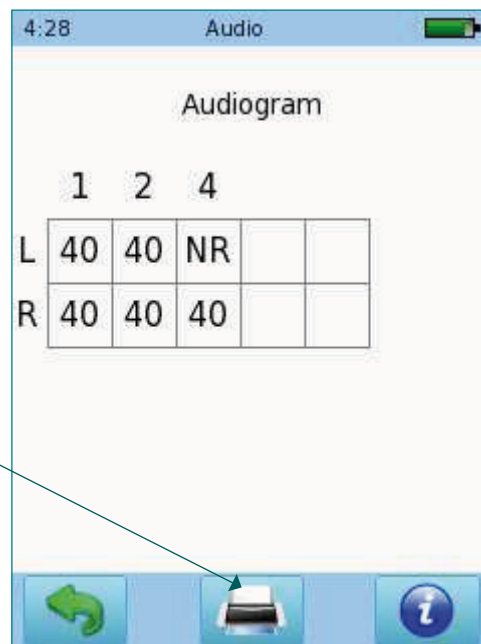

► Distortion Product Otoacoustic Emissions (DPOAE) Screening


Connect the probe to the red connector. Select either the Screening or the Standard button from the start menu.

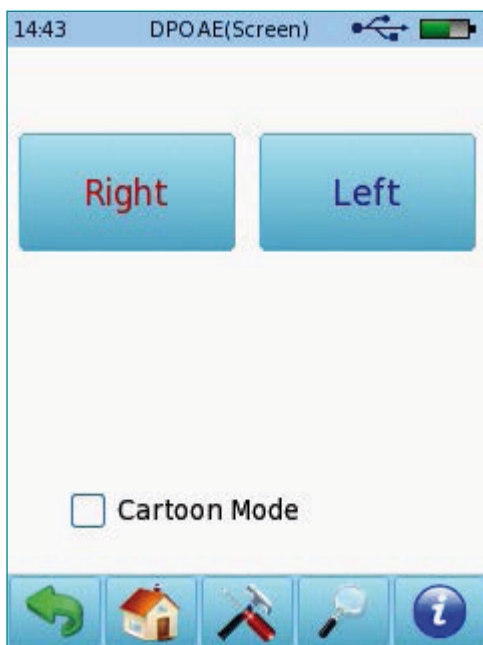

Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.


Select the test ear by pressing either the Right or Left button. Select cartoon mode to show balloons when the test is finished.

A calibration will be performed prior to measurement. A green light indicates successful calibration (see below left). After successful ear probe calibration, the test will start automatically.

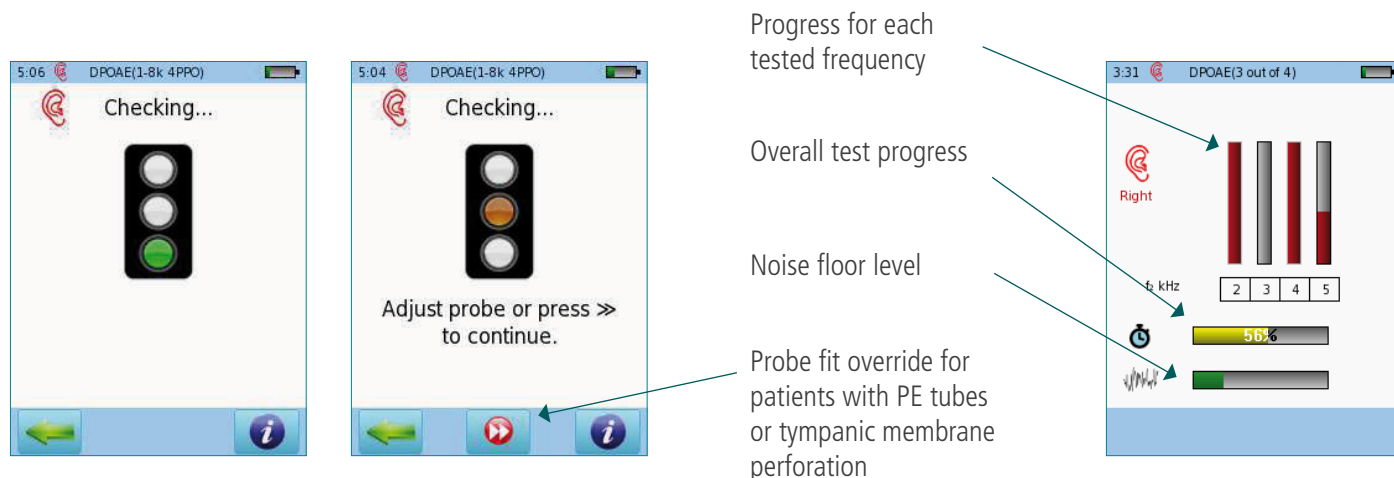


DPOAE test results Results view (Simple)

Please note that the initial result screen depends on the module settings in "DPOAE Preferences"

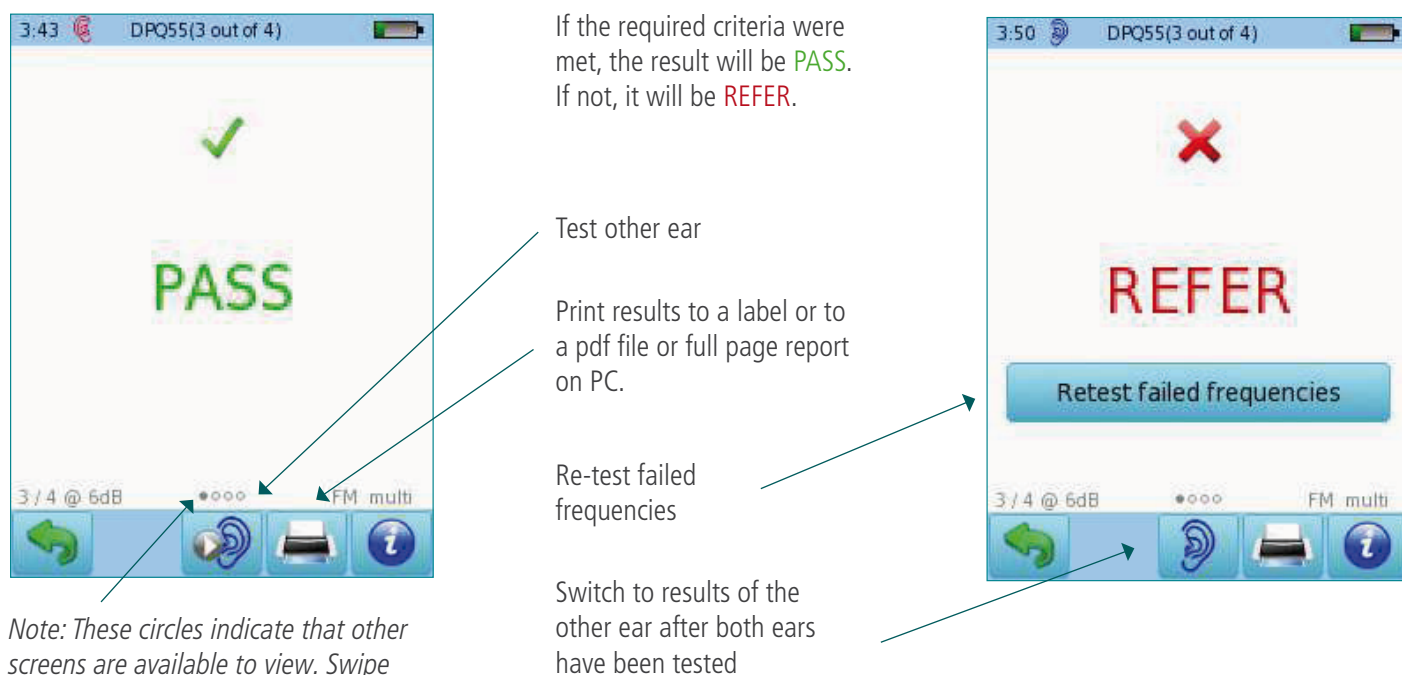


Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views

Detailed results view

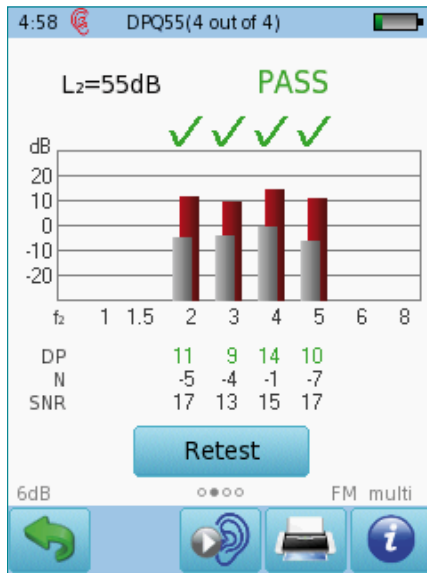

Red or blue bars indicate response, grey bars indicate noise

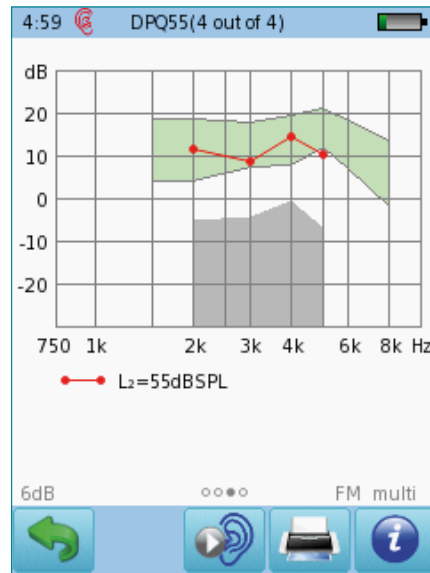

Outcome for each test sorted by f_2 and L_2



Numeric values shown in table format

Quick Guide Information

Release date: 2017-01-11

Revision: 02

Valid from: Firmware Rev. 2.1, Bio-link PC Software Rev. 2.0

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please contact your local Bio-logic distributor for further information.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

Legal manufacturer of the Bio-logic devices:

PATH medical GmbH

Landsberger Straße 65

82110 Germering

Germany

Tel: +49 89 800 76 502

Fax: +49 89 800 76 503

Internet: www.pathme.de



0124

Healthcare solutions with one thing in mind. You.

©2020 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. **7-26-1384-EN Rev01**

natus®

Natus Medical Incorporated

natus.com